

medw|rk®  
for mindful people

# ROCKSTAR

#safe



Intuitive operation  
through All-in-One System

twist ´ n ´ CATCH basket for  
fast and reliable retrieval

## The right tool for every situation

No matter how large the rock may be, the Rockstar is fearless: with the new stone remover for lithotripsy you are fully prepared for retrieving gall stones.

The guide wire lumen lets you reach the location of the stone reliably and the innovative operating concept allows intuitive breaking up of concrements if required.

Product code	Order number	Characteristics	ø Basket	Length	Accepted Guide Wire	Unit
② BAS3-K2-35-35-200	502486	twist ´ n ´ CATCH, integrated lithotripsy function, suitable for guide wire	35mm	200 cm	.035"	2

## How come the distal part of the Rockstar is stiffer compared to conventional baskets?


To fulfil our customers' needs we decided to offer our system short-wire compatible. Therefore, we attached a small guide wire duct at the distal tip helping to make cannulation a lot easier for you. This causes the tip to be a little bit stiffer.

## How do I hold the Rockstar in order to perform a lithotripsy?


To reach the highest level of force, get a firm grip on the thumb ring by putting the palm of your hand on the ring and holding on to it with your fingers. Now you can apply force by turning the wheel with your other hand (see pictures and video below).



Basket closed

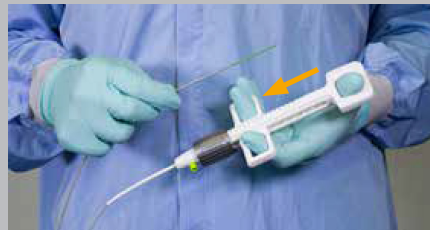

Basket opened

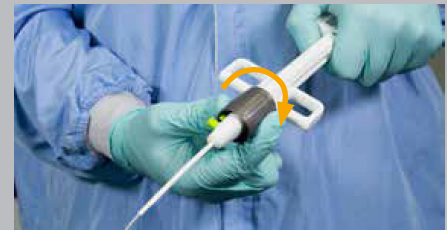

Mechanical Lithotripsy

## Competition Matrix

	Lithotripsy Baskets			Lithotripters	
	ROCKSTAR	Competitor C	Competitor B	rotaCRUSH	Competitor O
Basket Shape	Spherical	Hexagonal	Hexagonal	Spherical	Octagonal
Basket Size	Max. 35 mm	20 / 30mm	15 / 20 / 25 / 30mm	Max. 35mm	V-system: 26 / 30mm Standard: 30mm
Wire material	Compacted Wire Technology	Stainless steel	Stainless steel	Compacted Wire Technology	Stainless steel
Breaking / Safety Point	Yes	Yes	Yes	Yes	
Rotatable	Automatic	No	No	Automatic	Yes
Catheter Diameter	3,8 mm	N/A	N/A	2,35mm / 7fr	
Working Length	2000mm	2080mm	1910mm	2000mm	1950mm
Catheter Material	-	N/A	N/A	PTFE	PTFE
Contrast Injection	Yes	Yes	Yes	Yes	Yes
Coil Diameter	2.0mm 2.66mm	8F 2.18mm	6.5F 3.5mm	10.5F 3,5 mm	10,5 F
Coil Material	Coated stainless steel	Coated stainless steel	Coated stainless steel	Stainless steel	Stainless steel
Crown	Yes	No	No	Yes	Yes
Handle	Plastic, 2 moving parts	Plastic, 2 Instruments	Plastic, 2 Instruments	Metal, 1 moving part	Metal, 3 moving Parts
Wire Guided	Yes	Yes	Yes	No	Standard: No V-System: Yes
Separate Parts	1 unit	2 parts	2 parts	1 unit	Standard: 4 Parts V-System: 2 Parts
Disposable	Yes	Yes	Yes	Yes	No / Yes
Emergency Lithotripsy	Yes	Yes	Yes	Yes	Yes

Confidential - for internal use only